# Surgical Kit

Torque Ratchet

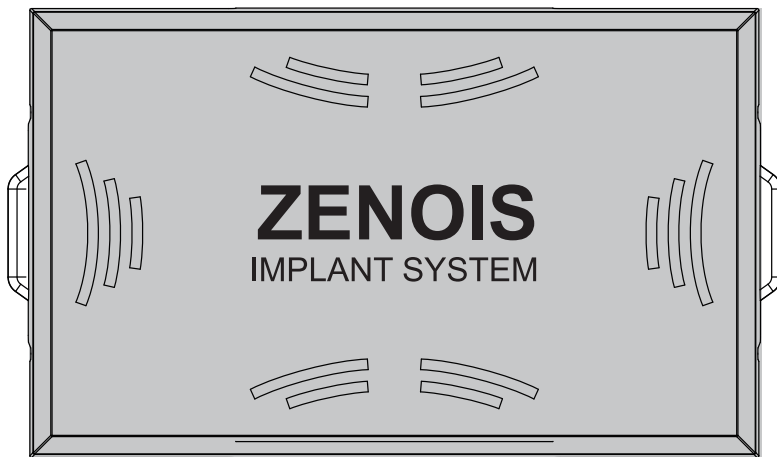
Lance Drill

Lindemann Drill



Step Drill

Tissue Punch

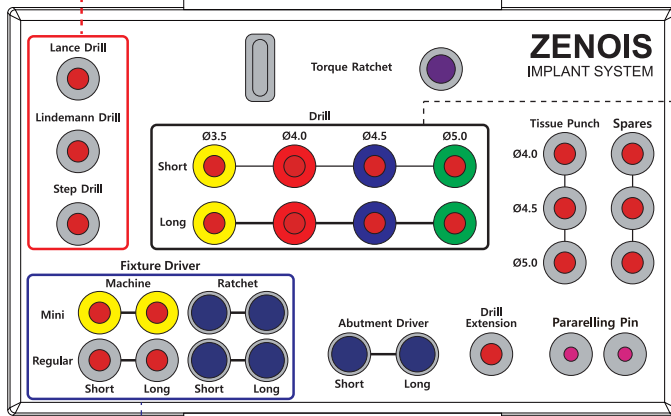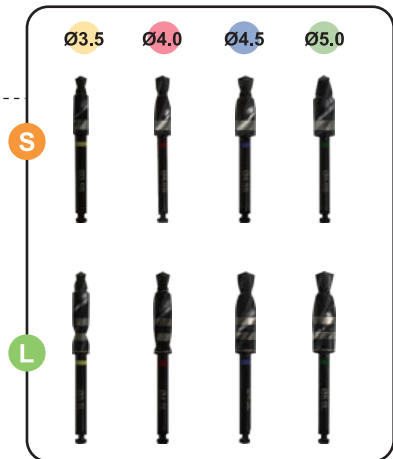Drill


Fixture Driver

Abutment Driver

Driver Extension

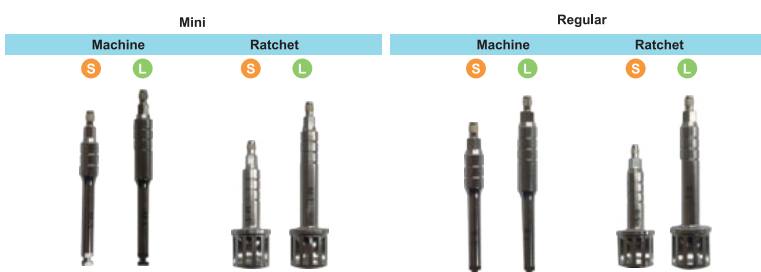Pararelling pin

## Lance Drill

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Used for first drilling and cutting the bone slightly by 2~3mm  
Deciding the bone density by drilling  
Hand piece speed: 1,000~1,500RPM

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Laser marking, regulating the length (except for 2,0/3,0)  
Drilling may be optional, depending on the bone density.  
The larger the diameter of drill, the smaller RPM  
Hand Piece speed : 1,000~ 1,500RPM

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Drilling for various depth with the drill  
Drilling may be optional, depending on the bone density.  
The surface of the drill : tine-coated  
Hand Piece speed : 1,000~ 1,500RPM

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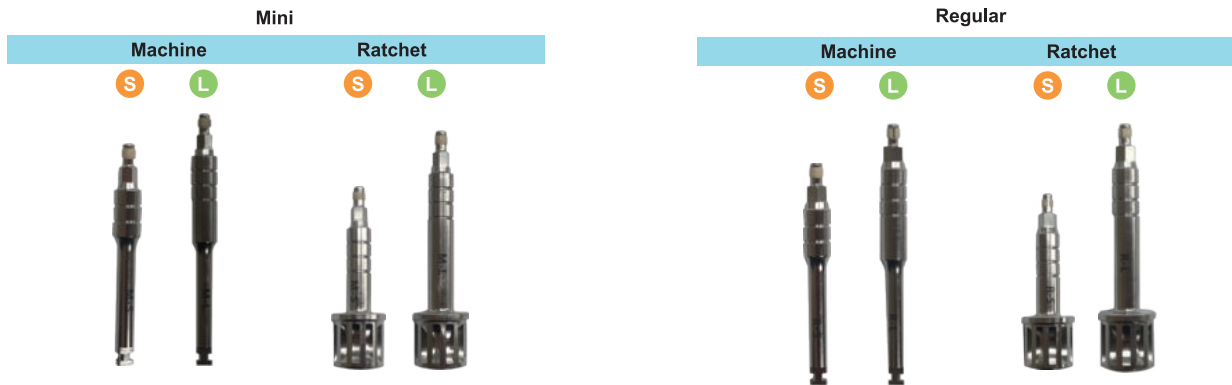
Analyzing the type of hole when it is drilled  
Planning for next surgical procedure  
Inserting into the 2.0 hole,  
Checking for the first drilling direction and position

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Handpiece type & Ratchet type(torque screw driver)  
Long type & short type  
Fixture driver for handpiece : taking out the fixture,  
in the beginning of implanting  
Torque control : 40Ncm

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Using in connecting abutment and abutment screw  
Torque screw driver  
Torque control : 20Ncm(mini type), 30Ncm(Standard type)

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Extending the length of surgical kit for handpiece type

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Connecting torque screw driver  
Laser marking(15,25,35Ncm)  
Controlling the force